ClinCal® - Calibrator

Plasma Calibrator Set lyophilised / Plasma-Kalibrator-Set lyophilisiert

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Intended use:

ClinCal® Plasma Calibrators are used for calibration of the analysis-system. These lyophilised calibrators are based on human plasma and are available as a calibrator set with 4 levels, including a blank. After reconstitution the calibrators have to be prepared like patient samples in one series of analyses.

Reconstitution:

Add exactly 1.0 ml of HPLC-water to the vial and mix for 15 min. When all material is dissolved, the solution is ready to use.

Storage and stability:

This product will be stable until the expiration date when stored unopened at 2 - 8 °C.

After reconstitution the stability of the analyte is:

- 8 hours when stored at 15 - 30 °C
- 7 days when stored at 2 - 8 °C
- 30 days when stored below -18 °C (avoid repeated freezing and thawing)

Notes:

The mean values of the analyte are chosen in a way that a linear calibration curve can be generated over a suitable range of concentration. According to quality assurance all ClinCal® Calibrators have to pass strict quality control procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume (CV) is < 1 %.

Mean values:

The mean values have been established at RECIPE, according to the Guideline of the German Medical Association on Quality Assurance (Rili-BAEK), with statistical methods.

Pack size:

ClinCal® Plasma Calibrator Set
Level 0 - 3
4 x 1 x 1 ml, **order no.: MS99113**

Precautions:

The human plasma which was used for manufacturing the calibrators was tested for the following infectious markers and found negative: HIV1/2- and HCV-antibodies, Hepatitis B-surface antigen. Nevertheless, the plasma calibrators should be considered as potentially infectious and treated with appropriate care.

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Level 0

This blank-level was manufactured from the same pool of human plasma like the others, but it was not spiked with Mycophenolic Acid.

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RECIPE CHEMICALS + INSTRUMENTS GmbH