ClinChek[®] - Control**Whole Blood Control lyophilised / Vollblutkontrolle lyophilisiert****FOR VITAMINS B1, B2, B6 / FÜR VITAMINE B1, B2, B6****Intended use:**

ClinChek[®] Whole Blood Controls are used for internal quality assurance in clinical-chemical laboratories. These lyophilised controls are based on human blood and are available with mean values in the normal as well as in the pathological range. After reconstitution the controls have to be prepared like patient samples in one series of analyses.

Reconstitution:

Add exactly 2.0 ml of HPLC-water to the vial. To dissolve the material completely, we recommend a so-called roller mixer. A treatment of 30 to 60 minutes with such an instrument is sufficient.

Storage and stability:

This product will be stable until the expiration date when stored unopened at 2 - 8 °C in the dark.

After reconstitution the stability of the analytes is, when stored in the dark:

- 5 hours at 15 - 30 °C
- 1 day at 2 - 8 °C
- 10 days at below -18 °C (avoid repeated freezing and thawing)

Notes:

The concentrations of the analytes are chosen in ranges where valid results can be obtained. According to quality assurance all ClinChek[®] Controls have to pass strict quality control procedures during manufacturing. RECIPE guarantees the same stability and constitution for each vial of one lot. The variation of the filling volume (CV) is < 1 %. The average residual moisture of this batch is 0.66 %.

Mean values:

The mean values and confidence intervals have been established at RECIPE, according to the Guideline of the German Medical Association on Quality Assurance (Rili-BAEK), with statistical methods.

Pack size:

ClinChek[®] Whole Blood Control
Level I
10 x 2 ml, **order no.: 8874**
Level II
10 x 2 ml, **order no.: 8875**
Level I, II
2 x 5 x 2 ml, **order no.: 8876**

Precautions:

The human whole blood which was used for manufacturing the controls was tested for the following infectious markers and found negative: HIV1/2-, HBV- and HCV-antibodies, Hepatitis B-surface antigen, HIV1- and HCV-RNA, HBV-DNA (NAT). Nevertheless, the whole blood controls should be considered as potentially infectious and treated with appropriate care.

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The mean values have been assigned with the ClinRep® Complete Kits for Vitamin B1, B2 and B6.

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RECIPE CHEMICALS + INSTRUMENTS GmbH