ClinTest[®] - Standard

Test Solution for Hemoglobin Variants /

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Intended use:

ClinTest[®] Test Solutions are intended to check the daily performance of your HPLC - system. This Test Solution needs no preparation and can be injected directly into your analysing system (injection volume: 20 µl). After separation, please compare the retention times of the peaks with those of earlier analyses. If you run the Test Solution for the first time, you can compare your results with the chromatogram on the back of this data sheet. Besides retention times you are also able to check peak shape, integration and baseline with your test chromatogram.

Storage and stability:

This product will be stable until the expiration date when stored unopened at 2 - 8 °C. After opening the vial, the Test Solution is stable for 4 weeks at 2 - 8 °C.

Pack size:

ClinTest[®] Test Solution for Hemoglobin Variants: 5 x 3 ml, **order no.: 13114**

Precautions:

The human whole blood which was used for manufacturing this Test Solution was tested for the following infectious markers and found negative: HIV1/2- and HCV-antibodies, Hepatitis B-surface antigen. Nevertheless, the test solution should be considered as potentially infectious and treated with appropriate care.

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