ClinTest® - Standard

Test Solution for Vitamin B1 /

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for HPLC Assay in Whole Blood /

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REF	24014
LOT	1498
	2020-12

Analyte / Analyt	Concentration / Konzentration
Thiaminepyrophosphate (TPP) / Thiaminpyrophosphat (TPP)	----
Thiaminemonophosphate (TMP) / Thiaminmonophosphat (TMP)	----
Thiamine / Thiamin	----

Intended use:

ClinTest® Test Solutions are intended to check the daily performance of your HPLC - system. This Test Solution needs no preparation and can be injected directly into your analysing system (injection volume: 50 µl). After separation, please compare the retention times of the peaks with those of earlier analyses. If you run the Test Solution for the first time, you can compare your results with the chromatogram on the back of this data sheet. Besides retention times you are also able to check peak shape, integration and baseline with your standard chromatogram.

Storage and stability:

This product will be stable until the expiration date when stored unopened at 2 - 8 °C in the dark. After opening the vial, the ready-to-use Test Solution is stable for 4 weeks, when stored at 2 - 8 °C in the dark.

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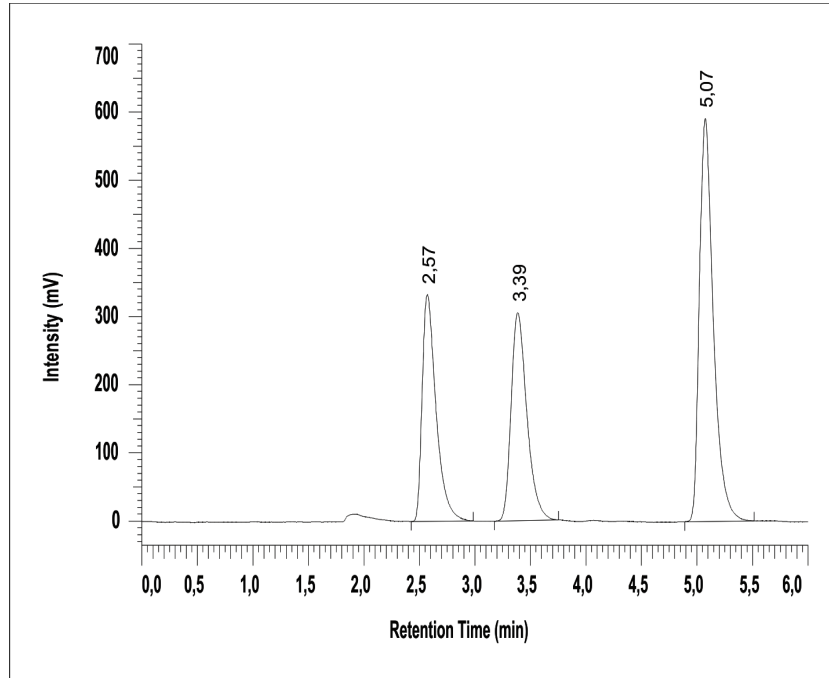
**Chromatogram of the Test Solution:
Musterchromatogramm der Testlösung:**

Thiaminepyrophosphate (TPP) / Thiaminpyrophosphat (TPP)	2.57 min
Thiaminemonophosphate (TMP) / Thiaminmonophosphat (TMP)	3.39 min
Thiamine / Thiamin	5.07 min